1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS Extraktionskit

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
Manual

1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS extraction kit

*For the extraction of 1,25-(OH)₂-vitamin D₃/D₂
from plasma and serum*

Valid from 2023-04-04

REF KM1120

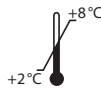IVD

Immundiagnostik AG, Stubenwald-Allee 8a, 64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: + 49 6251 70190-363

e.mail: info@immundiagnostik.com

www.immundiagnostik.com

Table of Contents

1. INTENDED USE	9
2. MATERIALS PROVIDED	9
3. MATERIALS REQUIRED BUT NOT SUPPLIED	10
4. PREPARATION AND STORAGE OF REAGENTS	10
5. SAMPLE PREPARATION	11
6. WARNINGS AND PRECAUTIONS	12
7. DISPOSAL	12

1. INTENDED USE

The 1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS extraction kit is intended for the sample preparation of 1,25-(OH)₂-Vitamin D₃/D₂ for the 1,25-(OH)₂-Vitamin D₃/D₂ LC-MS/MS application by means of extraction from human serum and plasma via immunoaffinity enrichment (unfilled ImmuPlate®). It is for manual and automatic use by laboratory professionals.

2. MATERIALS PROVIDED

Cat. No.	Identifier	Kit components	Quantity
KM0003	WASHSOL	Wash solution	2 x 80 ml
KM0005	RMMAT	Sealing mat for ELUPLATE	1 piece
KM0006	IPMAT	Sealing mat for PLATE	1 piece
KM1100	ELUSOL	Elution solution	2 x 15 ml
KM1110	ELUPLATE	Elution plate	1 piece
KM1120	PLATE	96-well plate	1 piece
KM1120	SLU	Antibody slurry for extraction of 1,25-(OH) ₂ -vitamin D ₃ /D ₂	1 x 10.5 ml

For reorders of single components, please use the catalogue number followed by the identifier without space as product number.

3. MATERIALS REQUIRED BUT NOT SUPPLIED

The following material is required for the 1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS extraction (not included in the kit):

- Glass tubes (inner diameter 10 mm)
- 96-well plate
- Precision pipettors and disposable tips to deliver 10–1 000 µl
- Centrifuge, capable of 550 g for 96-well plate
- Vortex mixer
- Vacuum centrifuge or nitrogen distributor
- Orbital shaker
- 96-well plate

The following accessories, among others, can be ordered separately for the 1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS application at Immundiagnostik AG:

- 1,25-(OH)₂-vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS kit (KM1020)
- calibrators and controls (KM1000CAL1–6 and KM1000CTRL1–3)
- tuning solution for 1,25-(OH)₂-vitamin D₃/D₂ (KM1000TU)
- tuning solution for the internal standard (KM1000TS)
- UPLC column (KM1000SP)
- in-line filter (KM1000IF)
- in-line filter holder (KM1000IH)
- wash plate (KM1120WASHPLATE)

Please ask for our single component price list.

Immundiagnostik AG offers further options for derivatization and sample preparation for the 1,25-(OH)₂-vitamin D₃/D₂ LC-MS/MS application. Please contact us for your individual solution.

4. PREPARATION AND STORAGE OF REAGENTS

The test reagents should be stored protected from light, dry and their specified storage temperature (2–8°C). The test reagents stored in this way are usable until the indicated expiry date.

5. SAMPLE PREPARATION

Serum and plasma samples are suited for the assay.

The samples must be centrifuged before use (minimum 5 min at 10 000 *g*).

Control samples should be carried along with each run.

Prior to use in the assay, allow all samples and reagents to come to room temperature (18–26 °C).

Mix samples and reagents well before use.

1.	Cut off approximately 5 mm of the outlet side of a 200 µl pipette tip and pipette 100 µl SLU into each PLATE well to be used. Swirl the bottle SLU regularly to prevent the slurry from settling.
2.	Add 500 µl sample into the corresponding wells of the filled PLATE.
3.	Seal the PLATE securely with IPMAT (use roller), invert a few times and incubate for 1 h at room temperature in an orbital shaker (1 200 rpm).
4.	Place PLATE on a 96-well plate (wash plate) and centrifuge at 550 <i>g</i> for 2 min so that no suspension remains on the IPMAT.
5.	Remove and discard IPMAT. Leave PLATE on wash plate, centrifuge 2 min at 550 <i>g</i> until dryness; discard the flow-through.
6.	Pipette 500 µl WASHSOL into each used well of the PLATE and centrifuge at 550 <i>g</i> for 2 min until dryness; discard the flow-through. Carry out this wash step three times in total.
7.	Place glass tubes in ELUPLATE, then place PLATE on ELUPLATE.
8.	Add 250 µl ELUSOL into each well used, centrifuge at 550 <i>g</i> for 2 min, and collect the eluate containing the analytes in the glass tubes.
9.	Evaporate the eluate under a nitrogen stream at 37 °C or in a vacuum centrifuge.
10.	Dissolve the residue in 175 µl solvent (adjust solvent to mobile phase).


6. WARNINGS AND PRECAUTIONS

- This product has been produced and placed on the market in accordance with the regulation (EU) 2017/746 (IVDR).
- ImmuPlate® is a brand of Immundiagnostik AG.
- All reagents in the kit package are for *in vitro* diagnostic use only.
- The assay should always be performed according the enclosed manual. Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. Immundiagnostik AG can therefore not be held responsible for any damage resulting from wrong use.
- Do not mix different lot numbers of any kit component. Plugs and caps of different reagents should not be swapped.
- Reagents should not be used beyond the expiry date stated on the kit label.
- Serious incidents are to be reported to Immundiagnostik AG and the national regulatory authorities.
- Please contact Immundiagnostik AG if one or more components of the kit are damaged, missing (see material supplied) or precipitates are visible in the ready-to-use solutions.
- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to Immundiagnostik AG along with a written complaint.
- The GHS symbols indicated on the individual components and specifications of the material safety data sheets (available on request from Immundiagnostik AG) must be noted. When working with these reagents, the legal protective precautions must be adhered to.

7. DISPOSAL

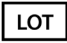Elution solution (ELUSOL) must be disposed as non-halogenated solvents.

Used symbols:

	Temperature limitation		Catalogue number
	In Vitro Diagnostic Medical Device		To be used with
	Manufacturer		Contains sufficient for <n> tests
	Lot number		Use by
	Do not re-use		Consult instructions for use
	Consult specification data sheet		

Immundiagnostik AG

Stubenwald-Allee 8a
64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: +49 6251 70190-363

info@immundiagnostik.com

www.immundiagnostik.com