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

*For the extraction of 1,25-(OH)₂-vitamin D₃/D₂
from plasma and serum*

Valid from 2024-05-06

REF KM1122

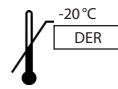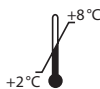IVD CE

Immundiagnostik AG, Stubenwald-Allee 8a, 64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: + 49 6251 70190-363

e.mail: info@immundiagnostik.com

www.immundiagnostik.com

Safety information

The assay has to be performed exclusively according to the instructions for use enclosed with the kit. Important safety information for this product can be found in chapter 6.

Table of Contents

1. INTENDED USE _____	10
2. MATERIALS PROVIDED _____	10
3. MATERIALS REQUIRED BUT NOT SUPPLIED _____	10
4. PREPARATION AND STORAGE OF REAGENTS _____	11
5. SAMPLE PREPARATION _____	11
6. WARNINGS AND PRECAUTIONS _____	13
7. DISPOSAL _____	14
8. SYMBOLS _____	14

1. INTENDED USE

The 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuPlate® DIY LC-MS/MS extraction kit is intended for the sample preparation of 1,25-(OH)₂-Vitamin D₃/D₂ for the 1,25-(OH)₂-Vitamin D₃/D₂ application by means of extraction from human serum and plasma via immunoaffinity enrichment (unfilled ImmuPlate®) and derivatization (IDKenhance®). It is for manual and automatic use by laboratory professionals. For *in vitro* diagnostic use only.

2. MATERIALS PROVIDED

Cat. No.	Identifier	Kit components	Quantity
KM0003	WASHSOL	Wash solution	2 x 80 ml
KM0005	RMMAT	Sealing mat for ELUPLATE	1 piece
KM0006	IPMAT	Sealing mat for PLATE	1 piece
KM1002	DER	Derivatization reagent	2 x 3 ml
	STOP	Stop solution	2 x 1.5 ml
KM1100	ELUSOL	Elution solution	2 x 15 ml
KM1110	ELUPLATE	Elution plate	1 piece
KM1120	PLATE	96-well plate	1 piece
	SLU	Antibody slurry for extraction of 1,25-(OH) ₂ -vitamin D ₃ /D ₂	1 x 10.5 ml

For reorders of single components, please use the catalogue number followed by the identifier without space as product number.

3. MATERIALS REQUIRED BUT NOT SUPPLIED

The following material is required for the 1,25-(OH)₂-Vitamin D₃/D₂ ImmuPlate® DIY LC-MS/MS extraction (not included in the kit):

- 96-well plate
- Precision pipettors and disposable tips to deliver 10–1 000 µl
- Centrifuge, capable of 550 g for 96-well plate
- Vortex mixer
- Vacuum centrifuge or nitrogen dryer
- Overhead shaker or orbital shaker
- 96-well plate

The following accessories, among others, can be ordered separately for the 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuPlate® DIY LC-MS/MS application at Immundiagnostik AG:

- 1,25-(OH)₂-vitamin D₃/D₂ IDKenhance® ImmuPlate® DIY LC-MS/MS kit (KM1022)
- calibrators and controls (KM1002CAL1–6 and KM1002CTRL1–3)
- tuning solution for 1,25-(OH)₂-vitamin D₃/D₂ (KM1000TU)
- tuning solution for the internal standard (KM1000TS)
- UPLC column (KM1002SP)
- in-line filter (KM1000IF)
- in-line filter holder (KM1000IH)
- wash plate (KM1120WASHPLATE)

Please ask for our single component price list.

Immundiagnostik AG offers further options for sample preparation for the 1,25-(OH)₂-vitamin D₃/D₂ LC-MS/MS application. Please contact us for your individual solution.

4. PREPARATION AND STORAGE OF REAGENTS

The test reagents should be stored protected from light, dry and their specified storage temperature (DER: -20 °C; all others 2–8 °C). The test reagents stored in this way are usable until the indicated expiry date.

5. SAMPLE PREPARATION

Serum and plasma samples are suited for the assay.

The samples must be centrifuged before use (minimum 5 min at 10 000 g).

Control samples should be carried along with each run.

Prior to use in the assay, allow all samples and reagents to come to room temperature (18–26 °C).

Mix samples and reagents well before use.

1.	Cut off approximately 5 mm of the outlet side of a 200 µl pipette tip and pipette 100 µl SLU into each PLATE well to be used. Swirl the bottle SLU regularly to prevent the slurry from settling.
2.	Add 500 µl sample into the corresponding wells of the filled PLATE.
3.	Seal the PLATE securely with the IPMAT (use roller), invert a few times and incubate for 1 h at room temperature in an orbital shaker (at least 1 200 rpm; make sure that all wells are effectively mixed) or an overhead shaker (15–20 rpm).
4.	Put the PLATE on a 96-well plate (wash plate) and centrifuge at 550 g for 2 min for removal of any SLU from the IPMAT.
5.	Remove and discard IPMAT. Leave PLATE on wash plate, centrifuge 2 min at 550 g until dryness; discard the flow-through.
6.	Pipette 500 µl WASHSOL into each used well of the PLATE and centrifuge at 550 g for 2 min until dryness; discard the flow-through. Perform this washing step a total of three times.
7.	Place the PLATE on the ELUPLATE.
8.	Add 250 µl ELUSOL into each well used, centrifuge at 550 g for 2 min, and collect the eluate containing the analytes in the glass tubes.
9.	Evaporate the eluate under a nitrogen stream at 37 °C or in a vacuum centrifuge.
10.	Add 50 µl DER into each glass tube and incubate for 1 h at 1 200 rpm on a shaker in the dark (attention: reaction is light sensitive).
11.	Add 25 µl STOP into each glass tube and incubate for 2 min at 1 200 rpm on a shaker.
12.	Evaporate the eluate under a nitrogen stream at 37 °C or in a vacuum centrifuge.
13.	Add 175 µl of solvent (adjust solvent to mobile phase) in every used well, seal the ELUPLATE with the RMMAT and shake at 1200 rpm for 1 min.


6. WARNINGS AND PRECAUTIONS

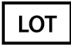- This product has been produced and placed on the market in accordance with the regulation (EU) 2017/746 (IVDR).
- IDKenhance® and ImmuPlate® are brands of Immundiagnostik AG.
- All reagents in the kit package are for *in vitro* diagnostic use only.
- The assay should always be performed according the enclosed manual. Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. Immundiagnostik AG can therefore not be held responsible for any damage resulting from wrong use.
- Do not mix different lot numbers of any kit component. Plugs and caps of different reagents should not be swapped.
- Reagents should not be used beyond the expiry date stated on the kit label.
- Serious incidents are to be reported to Immundiagnostik AG and the national regulatory authorities.
- Please contact Immundiagnostik AG if one or more components of the kit are damaged, missing (see material supplied) or precipitates are visible in the ready-to-use solutions.
- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to Immundiagnostik AG along with a written complaint.
- The GHS symbols indicated on the individual components and specifications of the material safety data sheets (available on request from Immundiagnostik AG) must be noted. When working with these reagents, the legal protective precautions must be adhered to.

7. DISPOSAL

Derivatization reagent (DER), stop solution (STOP) and elution solution (ELUSOL) must be disposed as non-halogenated solvents.

8. SYMBOLS

	Temperature limitation		Catalogue number
	In Vitro Diagnostic Medical Device		To be used with
	Manufacturer		Contains sufficient for <n> tests
	Lot number		Use by
	Contains plasma derivatives or human blood		Consult instructions for use
	Consult specification data sheet		Do not re-use
	Unique Device Identification		Contains material of animal origin
	Medicinal substance		Contains material of human origin

Immundiagnostik AG

Stubenwald-Allee 8a
64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: +49 6251 70190-363

info@immundiagnostik.com

www.immundiagnostik.com